



CHEMOFX®

A product of Precision Therapeutics, Inc.

PROCESS OVERVIEW

IN-OFFICE

1. Mark "Send to Precision for ChemoFx®" on physician order
2. For standing orders, call to inform your Precision sales rep a specimen will be sent for ChemoFx® testing
3. Provide patient with ChemoFx® patient brochure explaining their role in the process

OPERATING ROOM

Surgery

1. Ensure specimen cup (**without formalin**) or ChemoFx® shipping container is available prior to surgery
2. Collect tissue specimen
 - a. If sending tissue directly to Precision from operating room, collect specimen no smaller than 35 mg and no larger than 1 g. Package specimen according to instructions in ChemoFx® shipping container
 - b. If sending tissue to pathology, mark "Send to Precision for ChemoFx®" and immediately transport to lab

INTERVENTIONAL RADIOLOGY

Core Biopsy


1. Ensure specimen cup (**without formalin**) or ChemoFx® shipping container is available prior to biopsy
2. Collect biopsies
 - a. If sending tissue directly to Precision, provide the recommended number of biopsies:
 - 14g needle; ship a minimum of 2 cores
 - 16g needle; ship a minimum of 3 cores
 - 18g needle; ship a minimum of 4 coresPackage specimen according to instructions in ChemoFx® shipping container
 - b. If sending tissue to pathology, mark "Send to Precision for ChemoFx®" and immediately transport to lab

Fluid

1. Ensure evacuated container or ChemoFx® shipping container is available prior to procedure
2. Collect fluid specimen
 - a. If sending fluid directly to Precision, collect specimen no less than 100 cc and no larger than 500 cc. Package specimen according to instructions in ChemoFx® shipping container
 - b. If sending fluid to pathology, mark "Send to Precision for ChemoFx®" and immediately transport to lab

PATHOLOGY

1. Transfer tissue or fluid specimen to ChemoFx®-supplied specimen vial or fluid bottle
 - a. 35 mg minimum tissue specimen, 1 g maximum
 - b. 100 cc minimum fluid specimen, 500 cc maximum
2. Package specimen as quickly as possible after arrival and follow instructions in ChemoFx® shipping container

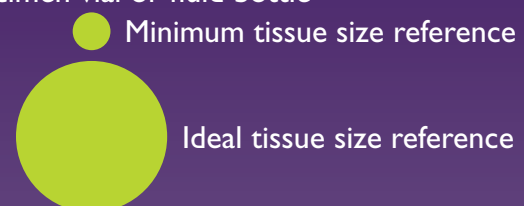

Contact Precision Client Services Team at 1-800-547-6165 to provide tracking information when sending a specimen